


## 1. Turning Device On, Off and Suspend.


Press power button for 2 seconds to turn device on. Device will beep before announcing it is ready. To suspend, press the power button once. To turn off, press the power button and the device will go into suspend mode and will prompt you to press the power button again to turn the device off. Note: OrCam will automatically suspend after 10 minutes (can be changed in settings menu).

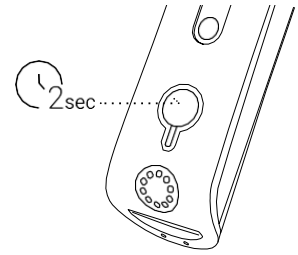


---

**2. Reading.** To initiate reading, tap the touch bar or use the finger point gesture. When using finger point gesture, ensure your finger is in the middle of the column and that you are pointing about a half inch below where you would like to start reading. When you hear a double beep, remove your finger from the text. To stop reading, tap the touch bar or use the hand gesture. If automatic detection is turned on, hold the item to be read in front of the camera for 3 to 5 seconds and OrCam will begin reading automatically.

**3. Touch Reading.** To browse through a document and read specific lines of text, use the finger gesture directly under the text you would like the device to read. OrCam will beep twice and begin reading 3 lines above you finger. To read a new section, slide your finger up or down without lifting your finger, to a new spot on the page. OrCam will double beep and begin reading the new section.

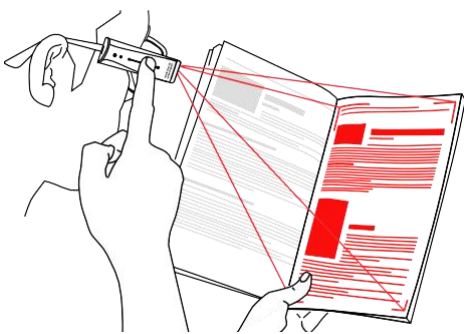


tap the touch bar

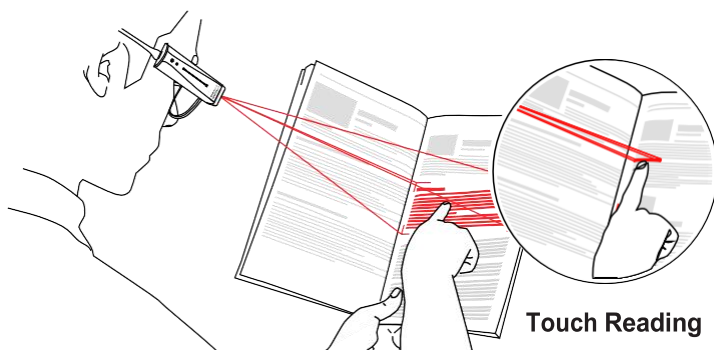

point to the page with nail facing forward


Touch Reading

**4. Reading Navigation.** To pause or resume reading, double tap the touch bar. To fast forward, swipe forward on the touch bar. To rewind, swipe backward on the touch bar.

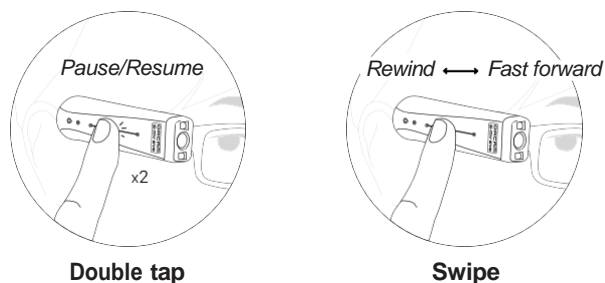


**5. Face Recognition.** Stand approximately 3 feet from the person and face them. Press and hold the touch bar for 15 to 30 seconds. You will hear multiple camera shutters. Ask the person to tilt or turn their head slightly. When you release your finger, you will be prompted to say the person's name after the beep. OrCam will repeat back the name you recorded and then prompt you to tap the touch bar to confirm or swipe to re-record the name.


If face recognition is set to manual, you must tap the OrCam to have the learned person announced. If face recognition is set to automatic, the OrCam will automatically announce any people who enter the camera's field of vision if they are saved to the device. If OrCam is set to fully automatic, learned faces will be announced and the device will announce the gender of the face in front of you for faces that have not been saved in the device.

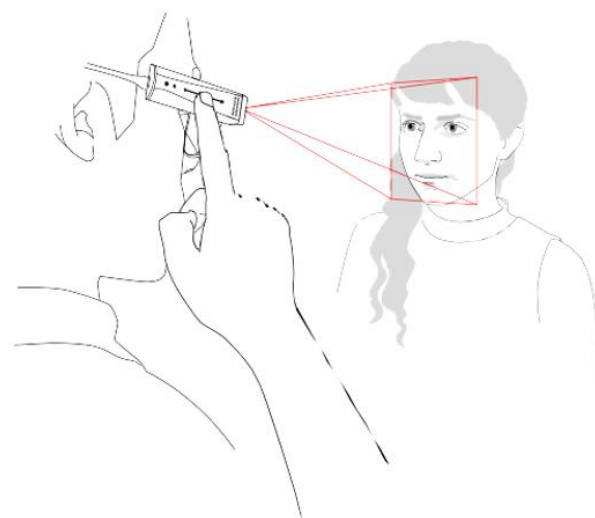

**6. Product Recognition.** Hold the product in your hand and scan/ find the barcode on the product. Hold the barcode in front of the device for 2-3 seconds until the OrCam states what the product is. If the OrCam says "unknown barcode", then use basic reading strategies to figure out what the product is.

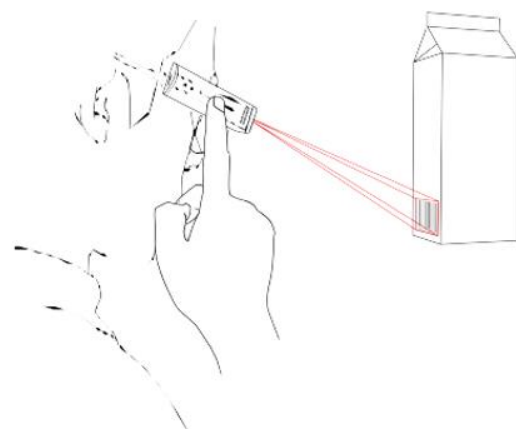


**7. Entering the Set-up menu.** While the device is not reading, double tap the device. You will hear a long beep. Then say “Enter User Menu” to access menu settings. Swipe forward or backward to cycle through options. Tap to select. Follow the prompts provided by the device. To back up one menu level, swipe backward until you hear “touch to go back to main settings menu” and then tap Touch bar. To exit the menu completely, double tap the touch bar.

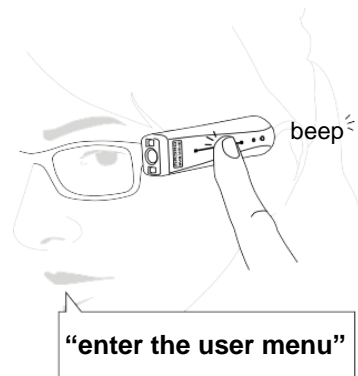

**8. Accessing the Time.** Bring the back of your wrist up in front of you as though you are looking at a wristwatch. Holding this position for more than 2 seconds will also cause the date to be read.



**9. Identifying currency.** Hold an unfolded bill in front of the camera about 12 inches away. After a few seconds, OrCam will automatically identify the bill for you. You can also tap the touch bar or use the finger point gesture to read the bill.

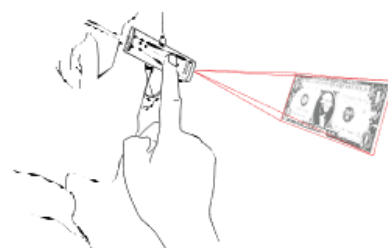

**10. Color Identifier.** Using the finger gesture, point at the color you would like identified and hold your finger there until OrCam tells you the color.

